Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

CE 0123

CONTEC

Ätzel

Etching gel

Gel de mordantage

Gel de ácido

Gel mordenzante

D
DENTAURUM

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Dear customer

Thank you for choosing a quality product from Dentaureum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use. The written instructions cannot cover all possible situations that can occur during use. In case of questions or ideas, please contact your local representative. As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the Internet at www.dentaureum.com, even though you may frequently use the same product.

1. Manufacturer

SAREMCO Dental AG | Gewerbestrasse 4 | CH-9445 Rebstein | Switzerland

2. Product description

CONTEC etching gel is for selective etching of dental enamel. The gel allows visual in-process control during etching. Contains phosphoric acid. Caustic!

3. Composition

Water, 35% phosphoric acid, gel former, coloring agent

4. Indication

For etching dental enamel for adhesive bonding of orthodontic elements (e.g. metal, ceramic or plastic brackets).

5. Contraindication

None when used purely for orthodontics.

6. Side effects

None known to date.

7. Interactions

None when used purely for orthodontics.

8. Application

8.1 Etching of teeth

Apply CONTEC etching gel evenly onto the dry enamel ridge. Leave for 30 seconds until small bubbles can be seen.

8.2 Rinse thoroughly.

8.3 Dry.

Note: Dental enamel will appear matt and whitish when adequately etched.

8.4 Please note!

If the tooth surface is contaminated (e.g. saliva) the etching process must be repeated (max. 10 seconds).

9. Storage

Storage temperature 4–28 °C (39–82 °F). If the room temperature exceeds 28 °C (82 °F), store the product in a refrigerator. Do not deep-freeze!

10. Precautions and hygiene

CONTEC etching gel contains phosphoric acid and is caustic. Avoid contact with skin, oral mucosa, eyes, clothes, metal etc. (we recommend both patient and operator wear protective goggles). Keep out of reach of children. The principles for safe personal and professional practice apply.

For dental use only.

11. Emergency procedures

Rinse thoroughly with water if the product comes into contact with the skin, oral mucosa or eyes. Consult an ophthalmologist if the product comes into contact with the eyes.

12. Batch number and date of expiry

For identification purposes in case of queries, please quote the batch number. The expiry date can be found on the packaging. The product should not be used after the expiry date.

13. Additional information

Should the user and/or the patient gain knowledge of serious incidents arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed.

14. Warranty

Our warranty is limited to the quality of our products. If the product quality is flawed, only the value of the product will be reimbursed. We accept no legal liability for further damages as a result of non-compliance with the Instructions for use or as a result of improper handling or improper use of the product. The user should check if the product is suitable for the case in hand before using the product. The user expressly accepts all risks involved in using the product and is solely responsible for any resulting damages.

15. Explanation of symbols on labels

 Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia |
Canada | USA | and in more than 130 countries worldwide.

DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

- ➔ Informationen zu Produkten finden Sie unter www.dentaurum.com
- ➔ For more information on our products, please visit www.dentaurum.com
- ➔ Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- ➔ Descubra nuestros productos en www.dentaurum.com
- ➔ Informazioni su prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour | Fecha de la información |
Data dell'informazione: 02/21

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications |
Reservado el derecho de modificación | Con riserva di apportare modifiche

D
DENTAURUM

Turnstr. 31 | 75228 Ispringen | Germany | Tel. +49 72 31/803-0
Fax +49 72 31/803-295 | www.dentaurum.com | info@dentaurum.com