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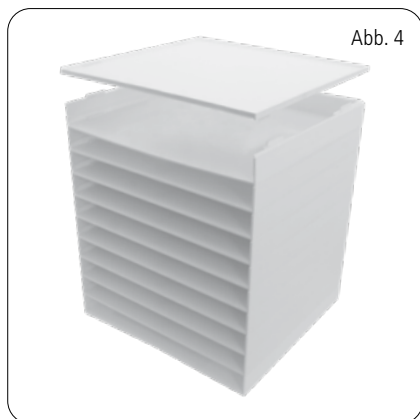
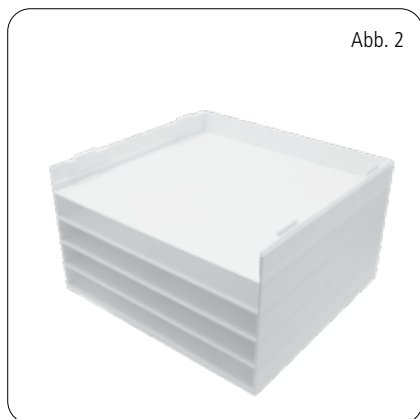
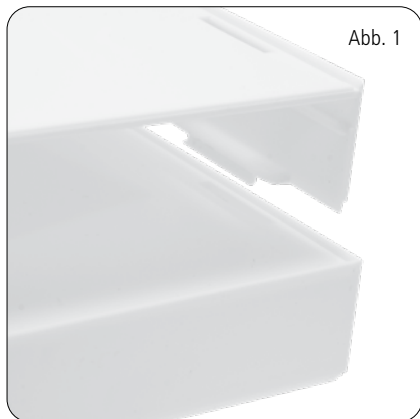
Dear customer,

Thank you for choosing a quality product from Dentaureum.

In order to achieve the best results with this product, it is important to carefully study and follow these Instructions for use.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaureum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaureum GmbH & Co. KG
Turnstraße 31 · 75228 Ispringen · Germany

2. General product description

The standard design of the cabinet for preformed bands, REF 890-414-00 consists of 10 drawers (without contents or vacuum-formed inserts).

The cabinet can be extended or reduced with further compartments. Please follow the assembly or disassembly instructions.

The compartments can be individually assembled and exchanged. See the corresponding list in the catalog or at www.dentaureum.com.

3. Extension of the cabinet with another compartment (Extension framework, REF 890-407-00)

- 3.1. Please remove all compartments and place the cabinet upside down. Remove the base. Carefully press the 4 pins aside until the base is released. (Fig.1 and 2)
- 3.2. Position the extension framework REF 890-407-00 so that the 4 pins snap into place and the extension framework is connected to the cabinet. (Fig. 3)
- 3.3. Replace the base on the extension framework until the 4 pins securely snap into place. (Fig. 4)
- 3.4. Place the cabinet back on the base and secure the stopper, REF 890-414-10, as with the others on the baseplate of the extension framework.
- 3.5. Replace the drawers back in the cabinet.

4. Remove one compartment from the cabinet.

- 4.1. Please remove all compartments and place the cabinet upside down. Remove the base. Push the 4 pins carefully aside until the base is released. (Fig. 1 and 2)
- 4.2. Remove the extension framework in the same way. (Fig. 3)
- 4.3. Replace the base on the extension framework until the 4 pins securely snap into place. (Fig. 4)
- 4.4. Place the cabinet back on the base and secure the stopper, REF 890-414-10, as with the others on the baseplate of the extension framework.
- 4.5. Replace the drawers back in the cabinet.

5. Availability

Please refer to the catalog or www.dentaureum.com for the delivery program of the cabinet.

6. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

7. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of the symbols used on the label REF 989-313-00).

Fig. 1

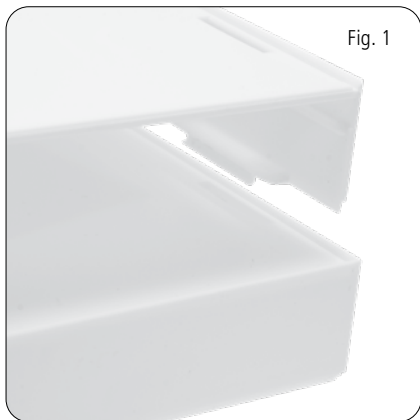
Fig. 2

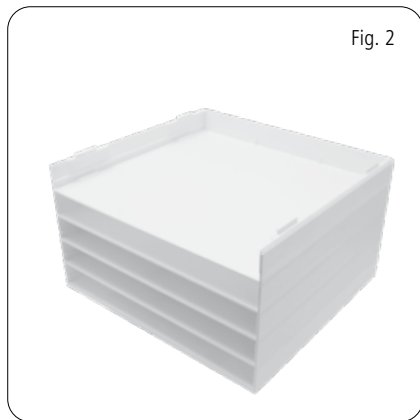
Fig. 3

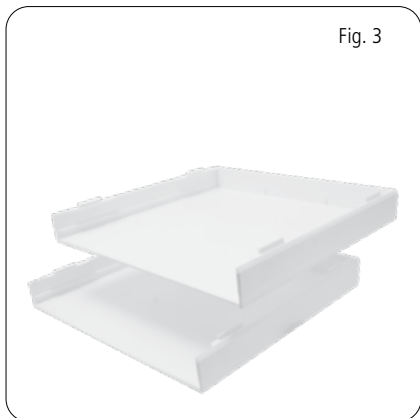
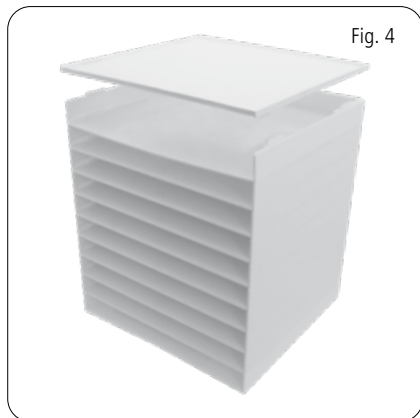
Fig. 4

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