Pflegeanleitung.
Care instructions.
Notice d'entretien.
Instrucciones de cuidado.

Deutsch3

English19

Français35

Español.....51

Liebe Patientin, lieber Patient,

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Dear Patient,

Congratulations on choosing treatment with prime4me® Aligners.

Now you have your own, individual transparent aligners that will correct your teeth, yet remain invisible. We have put together a set of instructions for you along with some important information to help you throughout your course of treatment. If you have any more questions, please contact the dentist who is treating you.

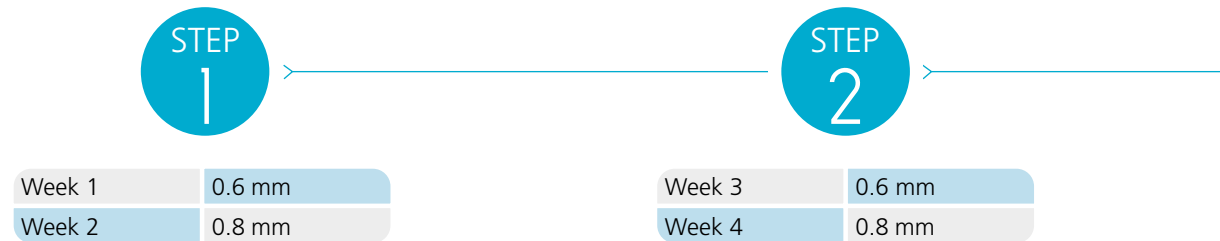
Enjoy your new smile!

Your prime4me® team

prime4me® Aligner concept.

The entire course of treatment consists of transparent aligners which gently move the teeth in the direction required. There are soft aligners (0.6 mm thick) and hard aligners (0.8 mm thick), each worn alternately for one week (7 days).

Begin with step 1. You will start with the soft aligners, then move on to the hard aligners. The aligners are individually packed and marked accordingly. Step 1 is followed by the aligners from step 2 and so on.

STEP
3

Week 5	0.6 mm
Week 6	0.8 mm

STEP
4

Week 7	0.6 mm
Week 8	0.8 mm

etc.

soft: foil thickness 0.6 mm

hard: foil thickness 0.8 mm

Using and storing.

Wear the aligners for at least 22 hours a day.

Store the aligners in the aligner box included in the delivery (for example, when you are eating or cleaning your teeth). Keep the aligners away from sources of heat as this may have a negative impact on their shape. This, in turn, means the aligners cannot move the teeth correctly.

Keep to all of the instructions and remember to attend all appointments with your dentist.

It may be necessary for attachments to be placed on your teeth during treatment. These support the teeth as they are moved spatially. If you lose an attachment during treatment, please contact your dentist immediately.

Please pay attention during the entire treatment to good oral hygiene. Clean your teeth and the spaces in-between thoroughly after every meal before you put the prime4me® Aligners back into place.

To be on the safe side, keep the aligners from the week before, preferably in their original packaging (backup aligners). You can use the backup aligners if you lose one for the current week or if it gets damaged. This will stop your teeth from moving back to their former position. Inform your dentist immediately so that replacement aligners can be made.

Inserting and removing.

Inserting the prime4me® Aligner.

Position the aligner on the back teeth first.

Then press the front part of the aligner down onto your teeth.

Removing the prime4me® Aligner.

First, remove the aligner from the back teeth.

Then remove the front part of the aligner.

Inserting and removing when you have attachments

Inserting the prime4me® Aligner.

Position the aligner on the front teeth first. Carefully slide the aligner over the attachments.

Then press the back part of the aligner onto your teeth.

ents on your teeth.

Removing the prime4me® Aligner.

Remove the aligner from your back teeth first. Be careful to pull gently around the attachments.

Then remove the front part of the aligner.

Cleaning and maintaining.

- 🔊 Clean the aligners each time before insertion.
- 🔊 **Recommendation:** Clean the aligners on the inside and the outside with a soft tooth brush using toothpaste with a low proportion of abrasive particles (low RDA, relative dentin abrasion value). Then rinse the aligners under cold running water.
- 🔊 **Caution:** Warm water may cause the aligners to deform. Aggressive soaps or tooth pastes with a high proportion of abrasive particles (high RDA value) may damage the surface of the aligners.
- 🔊 Clean your teeth and the spaces in-between thoroughly after every meal and before you put the prime4me® Aligners back into place.
- 🔊 Once daily and whenever the aligners are hard to get clean, you can use a special cleaning powder, e.g. CETRON®, or ask your local pharmacy what they recommend.

Eating and drinking.

-  Remove the aligners before you eat and place them in the aligner box. When you are not wearing the aligners, you can eat whatever you want.
-  When you are wearing the prime4me® Aligners ...
 - ... don't chew chewing gum → it'll stick to the aligners
 - ... don't drink any warm drinks like tea or coffee → they will deform the aligners
 - ... don't eat or drink anything containing sugar or acid → this may cause caries
 - ... don't smoke → this may discolor the aligners and cause periodontal damage
-  Clean your teeth and the spaces in-between thoroughly after every meal and before you put the prime4me® Aligners back into place.

FAQ.

The aligners don't fit properly and are hurting. What should I do?

It is normal for you to notice a slight feeling of pressure at the beginning of the treatment and whenever you first use a new aligner. This should subside after a few days. It's a sign that the teeth are moving in the right direction. If, however, an aligner doesn't fit at all and the pain doesn't go, please arrange to see your dentist immediately.

Can the aligners be cleaned with warm or hot water?

No. Please only clean and rinse the aligners with cold water. Warmth may cause the aligners to deform and then no longer fit properly.

Will the aligners affect how I speak?

No. As with any form of orthodontic treatment, you will have to get used to the feeling of wearing the aligners. Any problems you may have with speaking will soon disappear after a short while.

Can I really eat and drink whatever I want?

As long as you are not wearing the aligners, there are no restrictions. Clean your teeth and the spaces in-between thoroughly after every meal and before you put the prime4me® Aligners back into place. You should not eat anything whilst you are wearing the aligners. Please drink cold water only. Eating or drinking anything containing sugar or acid whilst wearing the aligners may cause caries, gum disease or discoloration of the teeth and the aligners.

What should I do if I lose an aligner or one gets damaged?

To be on the safe side, keep the aligners from the week before, preferably in their original packaging (backup aligners). You can use the backup aligners if you lose one for the current week or if it gets damaged. This will stop your teeth from moving back to their initial position. Inform your dentist immediately so that replacement aligners can be made.

Instructions for use

Dear customer,

Thank you for choosing a quality product from Dentaaurum. Please read these instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

1. Manufacturer

Dentaaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General device description

prime4me® Aligner are patient-specific transparent dental splints. The prime4me® Aligner system is based on a treatment concept with two aligners with a prescribed order in which the splints are to be worn. The prescribing dentist defines the necessary treatment steps in the treatment plan. You can find more information at www.prime4me.com.

3. Intended purpose

prime4me® Aligner are used for the orthodontic treatment of tooth misalignments in patients.

4. Intended users

All products are intended for dental professionals and for patients, for whom a custom product has been prescribed.

5. Indications

The following lists examples of typical indications: crowding, gap closure, deep overbite, open bite, crossbite, midline correction.

6. Contraindications

The following lists examples of typical contraindications: patients with periodontitis, bridges, implants or devitalized teeth.

7. Patient target group

All patients with permanent, erupting or mixed dentition and adult patients with permanent dentition for whom a removable appliance can be placed within the framework of orthodontic treatment.

8. Composition

The material composition is included in the orthodontics materials list; please refer to the catalog or www.dentaaurum.com.

9. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components.

It is the user's responsibility to decide on combining different products with regard to the compatibility of chemical, physical and mechanical properties. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

10. Further information

Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed accordingly.

11. Disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

12. Product range

For the complete product range of prime4me® Aligner, please see our orthodontics catalog or www.dentaurum.com.

13. Quality

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences.

14. Explanation of symbols used on the label

Please refer to the label. These Instructions for use and additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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