




passion and
precision.

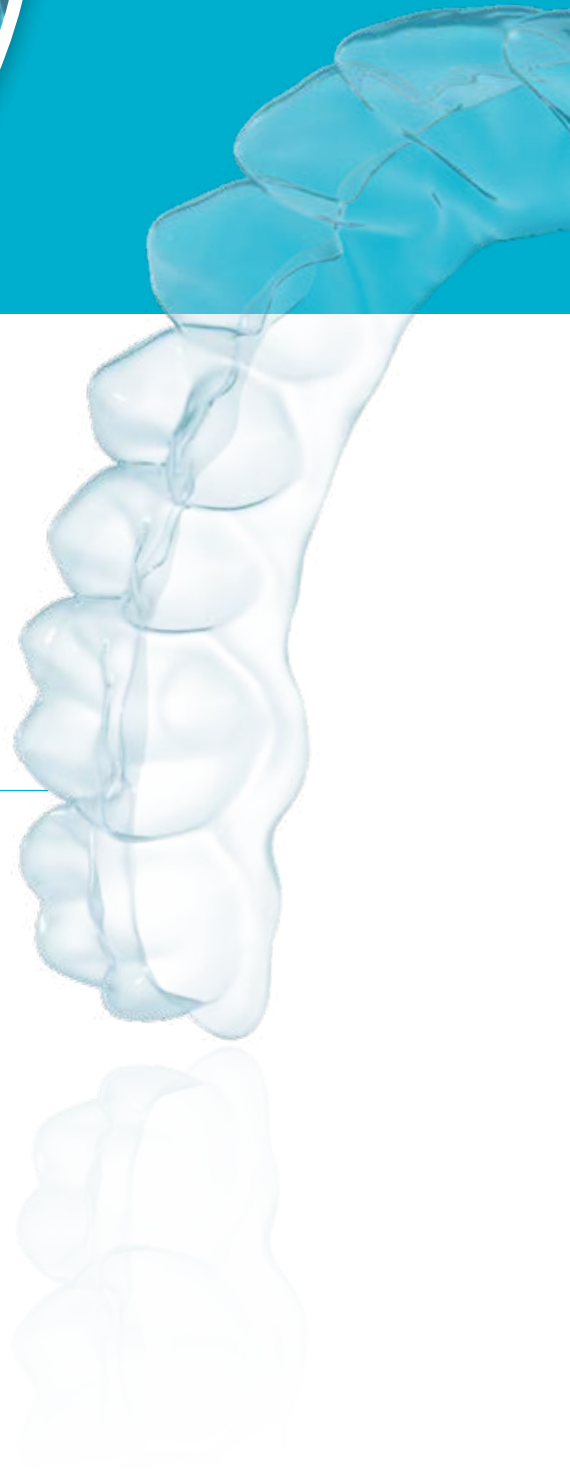


 Transparent aligner material
for almost invisible therapy.



prime4me® Aligner.

Innovative splint concept for the best treatment results.

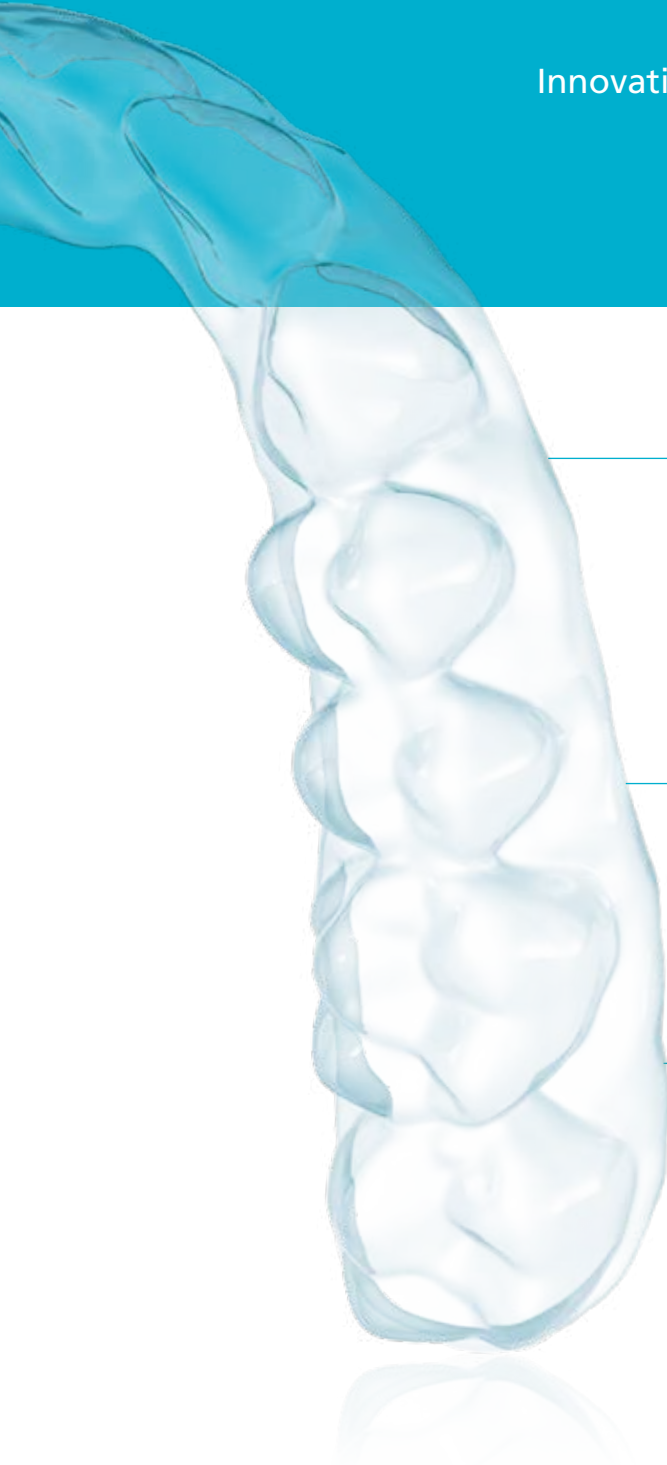


Aligner covers part of the gingiva for better transfer of forces.

Digital case planning with OnyxCeph^{3™}.

Personal consultation provided by an experienced support team.

Treatment concept with 2 aligners.

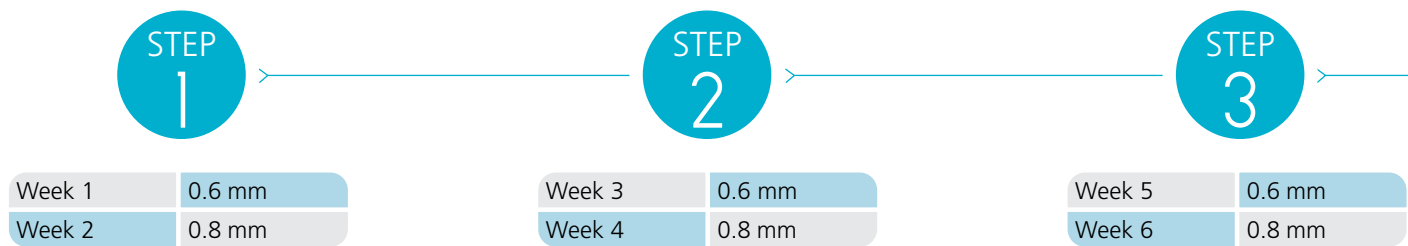

Advantages.

- Optimal transfer of forces by using two foil thicknesses: soft (ø 0.6 mm) and hard (ø 0.8 mm).
- Movement of up to 0.5 mm per step possible: Period of treatment can be adapted optimally to each individual situation.
- Gentle, careful aligner therapy; movement is initiated by the soft foil. The movement is then continued and stabilized with the hard foil. Tooth movement is therefore continual and gentle in each step.
- Patient compliance is higher thanks to the weekly change of aligners: patients start each week with fresh, new aligners.
- Weekly change has positive effects: less discoloration and plaque on the aligner surface, yet full transfer of forces.
- Including 3D Viewer free of charge for patient consultation and to view the treatment proposal. Example, see www.prime4me.com

STEP
4

Week 7	0.6 mm
Week 8	0.8 mm

STEP
5

Week 9	0.6 mm
Week 10	0.8 mm

etc.



soft: foil ø 0.6 mm



hard: foil ø 0.8 mm

Treatment concept with 2 aligners.




prime4me® Aligner can be used to treat many different types of tooth misalignment.


Possible indications are:

-  Crowding
-  Gap closure
-  Deep overbite
-  Open bite
-  Crossbite
-  Midline correction
-  Overjet-overbite correction


Come and talk to us: together we can find out if prime4me® Aligner is the right choice for your patient.

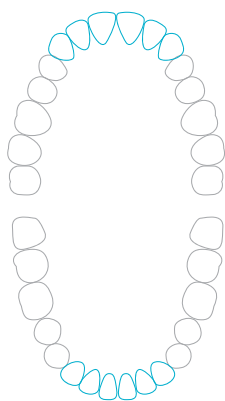

3 – 3

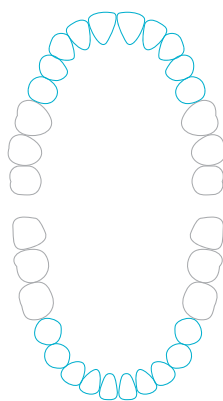

5 – 5

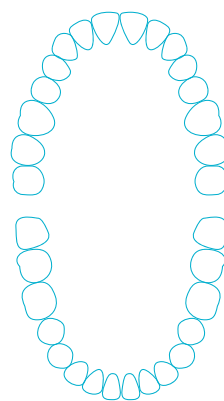

7 – 7

	smart	advanced	performance
Steps	up to 5	6 – 9	10 – 15
Duration	≈ 70 days ≈ 10 weeks	≈ 126 days ≈ 18 weeks	≈ 210 days ≈ 30 weeks
Aligners per jaw	2 – 10	12 – 18	20 – 30
Aligners in total	4 – 20	24 – 36	40 – 60
Deliveries	1	2	3

We work closely with you when planning the treatment. Our experienced personnel will take into consideration alterations you would like to make and you will receive a modified case plan very quickly for your approval.

The aligners will not be fabricated until you have sent your approval to the order platform. Should treatment be canceled after the planning stage, we will invoice only a flat fee* for treatment planning.

Progress is monitored by means of intermediate check-ups with intra-oral scans or model scans for the versions advanced and performance. Treatment planning can be modified if required. The aligners will be adjusted to the current situation, and then produced and sent to your practice. Our target-oriented aligner production saves resources and protects the environment.

* see price list

Attachments.

Our dental specialists will include attachments in the plan if your patient should require them. The attachment guides will be included in the delivery.



You can find a step-by-step guide on how to fit them in our download center.





More information
and accessories at:

www.dentaurum.com




Activation surfaces.

Interproximal enamel reduction.

In some cases it may be necessary to create interdental space so that tooth movement is possible. Mesial and/or distal interproximal enamel reduction (IPR) is possible. Detailed instruction on how much enamel should be removed from which tooth is enclosed in the delivery or it can be stored online with your order at digital.dentaurum.de.

Dentaurum Digital.

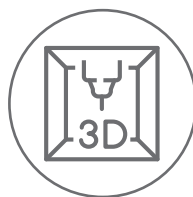

Innovative order platform for the products of tomorrow.



prime4me® Aligner:

innovative splint concept for the best treatment results.



prime4me® Model printing:

Base models and dental arch models for orthodontics.



prime4me® RETAIN3R:

Unique fit using a three-dimensional construction design.



Advantages.

-  Easy, intuitive ordering for all Dentaurum Digital products.
-  Easy file upload.
-  Display of current order status.
-  Filing of all open and closed orders.
-  Online order-related communication with an experienced support team via the order platform.



Discover now:
digital.dentaurum.de

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.



DENTAURUM
QUALITY
WORLDWIDE
UNIQUE



www.dentaurum.com



CUSTOMER SERVICE prime4me®

Customer Support Digital

+49 72 31 / 803-280 | digital@dentaurum.de

Order now:

digital.dentaurum.de



Date of information: 2023-02
Subject to modifications

D
DENTAURUM