Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo

Dublitop® micro

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Dear Customer

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaureum.com, even though you frequently use the same product

Manufacturer

Dentaureum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

With reference to our quality standards

Dentaureum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

Dublito[®] micro

The innovative portioned package for the microwave.

Precision duplicating material for combination, model-casting and acrylic-casting processes. Suitable for phosphate-bound investment materials such as rema[®] Exakt and rema[®] dynamic and ethyl-silicate investment materials.

Dublito[®] is not meant for use with plaster and plaster containing investment material. For those we recommend our special duplicating material Dubliplast[®] (5 kg, REF 165-500-00) and Dubliplast[®] liquid concentrate (2.7 kg, REF 165-550-00).

1. First Time Use

Before working with the material, make sure that you shake the beaker well!

Open beaker and place lid **loosely** on top.

Attention: A tightly closed beaker could explode!

Place beaker in the centre of the microwave.

When used for the first time, the material has to be heated quickly to the boiling point (100 °C/212 °F) otherwise it will not gel.

Attention: Do not allow to boil.

The following information pertains to a microwave with 800 W at full power. At higher Wattage it has to be changed to the equivalence of 800 W. At lower Wattage the melting time has to be increased.

The heating takes place in 2 stages:

1. Approximately 2 minutes 800 W (full power) remove beaker with Dublitolop® micro (**Caution: beaker is hot**) remove lid, stir with spatula and test for correct temperature.
2. Place lid **loosely** on beaker and return to the microwave for approximately 20–30 seconds, just to the boiling point.
Remove the beaker with Dublitolop® micro and test for the correct temperature (100 °C/212 °F).
Attention: beaker is hot.

2. Cooling to pouring temperature

Place the beaker containing Dublitolop® micro in a basin filled with cold tap water (water level equivalent to level of material in beaker) and stir the material with a metal spatula for about 4 minutes until it has reached the correct temperature for use (50 °C/122 °F). Check the temperature repeatedly with a thermometer.

Use only a clean metal spatula. Avoid trapping air bubbles in the material. Then allow the beaker to rest briefly to allow any bubbles to escape.

3. Reuse

Dublitolop® micro duplicating material which has been melted and then re-gelled must be cleaned carefully under running water. Cut it into small pieces with a knife and put it back into the beaker. Do not add water! Close the lid of the beaker tightly to store the Dublitolop material.

Before re-melting the material in the microwave, open the lid and lay it **loosely** on top of the beaker.

Danger! If closed tightly, the beaker may explode!

Place the beaker in the centre of the microwave oven.

When re-melting, Dublitolop® micro, the material has to be heated to 90–93 °C/194–199.4 °F.

Heating: approx. 2 minutes 800 W (full power).

Remove the beaker with Dublitolop® micro. **Danger! The beaker is hot.** Remove the lid, stir briefly with a spatula, check the temperature and ensure that the material contains no lumps.

If necessary, replace the lid **loosely** on the beaker and re-heat again for a few seconds. Do not allow it to boil!

Remove the beaker with Dublitolop® micro from the microwave and check the temperature (100 °C/212 °F).

Danger! The beaker is hot.

Cooling to pouring temperature

Cool manually in the water bath or with the Dublitolop® micro set as described above.

General Duplicating Instructions

1. Preparing the model

Use temperature-resistant blocking wax (e. g. self-adhering preparatory wax 0.5 mm, REF 120-025-00 and thermo wax, REF 120-170-00).

2. Moistening the plaster models

Clean the plaster models for duplicating and lay them in warm water (40–45 °C / 104–113 °F) for approximately 30 minutes with the teeth downwards.

3. Duplicating

Dab water off the parts and place the model in the centre of the flask. Recommended duplicating flask: Dentaureum metal duplicating flask, REF 165-700-00.

Pouring temperature 50 °C / 122 °F. Pour the duplicating material into one side of the flask. Allow the duplicating flask to stand for 20 minutes and then place it in a water bath (approx. 2 cm in height) at 18–25 °C / 64.4–77 °F.

Total solidification time until removal: approx. 40 minutes.

Important: For plastic vessels, the solidification times may be up to 30 % longer.

After this, the original model can be lifted carefully parallel out of the Dublitolop® micro duplicating gel (model-removal tongs, REF 008-030-00). Without any further standing time, the investment material is poured into the dry hollow mould (observe the prescribed mixing ratio). After about 40 minutes' setting time, the model can be peeled out.

Important: The duplicating mould and the mixed investment material should not be too cold (room temperature) otherwise the investment material will not harden properly.

4. Reuse of Dublitolop® micro

If handled properly, every beaker of Dublitolop® micro can be melted at least 10 times.

The quantity of 4 x 350 g Dublitolop® micro is therefore sufficient for at least 4 x 10 duplicating operations using the Dentaureum metal duplicating flask, REF 165-700-00.

When the elasticity begins to decline, use a fresh beaker of Dublitolop® micro duplicating material.

Important: Never mix used Dublitolop® micro material with fresh material.

5. Shelf life of Dublitolop® micro

24 months in closed original container.

6. Disposal

Dublitolop® micro duplicating material can be disposed of with normal domestic waste.

Dispose of empty packages in accordance with the local regulations.

7. Package sizes

Dublito[®] micro (REF 108-615-00):

4 x 350 g duplicating material (liquid) in plastic beakers.

8. Duplicating accessories

Duplicating flask, REF 165-700-00:

Non-oxidizing metal flask for short gelling times with holding device for casting sprue.

Model-lifting tongs, REF 008-030-00.

9. Possible problems and remedies

Problem	Cause	Remedy
Poor surface quality of Dublito[®] micro models.	Hollow duplicating mould too moist. Investment material mixing time too short. Non observance of investment material consistency. Non observance of investment material setting time. Setting delay caused by cold duplicating and investment materials.	Follow the instructions for processing.
Sudden decline in crack strength. Greasy consistency. Low accuracy.	Duplicating material is too old. Duplicating material overheated. Contamination by foreign material such as detergent, alum or electrolyte.	Material is permanently damaged. Discard.
Dublito[®] micro does not gel properly at first use.	Material not heated to boiling point for first use.	Heat material briefly to boiling point for first use.
Viscous Dublito[®] micro material.	Water evaporated, e.g. through microwave heating without lid.	Observe processing instructions. Used Dublito [®] micro material must be cleaned under running water. Lay lid on beaker when heating in microwave. Store only with lid firmly closed.

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