Gebrauchsanweisung | Instructions for use
Mode d'emploi | Modo de empleo

Sandstrahlgerät **Dentablast.**
Sandblasting machine **Dentablast.**
Sableuse **Dentablast.**
Arenadora **Dentablast.**

REF 094-530-00 | REF 094-531-00.

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1. Hersteller 3

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Contents

1. Manufacturer 11

2. With reference to our quality standards 11

3. Safety instructions..... 11

 3.1 Correct use..... 11

 3.2 Hazard warning symbols..... 11

 3.3 Warranty and liability..... 11

 3.4 General information on the safe use of this machine 12

 3.5 Disposal..... 12

 3.6 Packing 12

 3.7 Delivery 12

4. Areas of use and description 12

 4.1 Function 12

5. Installation..... 12

 5.1 Positioning the machine 12

 5.2 The power supply..... 13

 5.3 The compressed air supply 13

 5.4 Connection to an extraction unit 13

6. Operating the sandblasting machine Dentablast 13

 6.1 General information 13

 6.2 Setting the blasting pressure..... 13

 6.3 Re-filling the abrasive material 13

 6.4 Sandblasting..... 13

 6.5 Emptying the abrasive material 13

7. Maintenance..... 15

 7.1 Replacing the gloves..... 15

 7.2 Replacing the sandblasting nozzle 15

 7.3 Replacing the fluorescent lamp..... 15

 7.4 Replacing the viewing window 15

 7.5 Replacing the vacuum tube and vacuum pipe 15

8. Spare parts 15

9. Accessories 16

10. Technical data 16

11. Declaration of Conformity 17

Dear customer

Thank you for choosing a quality product from Dentaaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaaurum.de, even though you frequently use the same product.

1. Manufacturer

Dentaaurum GmbH & Co. KG | Turnstraße 31 | 75228 Ispringen | Germany

2. With reference to our quality standards

Dentaaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

3. Safety instructions

 **Caution** **Read through the instructions for installing and operating the unit carefully before switching it on.**

3.1 Correct use

The Sandblasting machine Dentablast is intended solely for homogeneous mixing of investment, plaster or silicone used for fabricating models in the dental laboratory. The unit should only be used for the purpose for which it is intended. Dentaaurum GmbH & Co. KG, is not liable for damages resulting from any other use of the unit. Approved use also includes adhering to the instructions for use as well as servicing and carrying out maintenance work at regular intervals.

When a Dentaaurum product is finally no longer in use the disposal regulations of that particular country apply. Dentaaurum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

3.2 Hazard warning symbols

The following symbols are used in the instructions to indicate risks:

 **Warning** **Indicates a potential danger to the life and health of the person. Disregarding this sign can critically affect a person's health and may even result in life-threatening injuries.**

 **Caution** **Indicates a potentially dangerous situation. Disregarding this sign can result in minor injury or damage to the equipment.**

 **Note** **Indicates general information on operating the unit.**

3.3 Warranty and liability

- Our general conditions of sale and delivery apply. Guarantee and liability claims for damages to persons or equipment are invalid if the damage is found to have resulted from any of the following:
- improper installation, operation, assembly or maintenance of the unit
 - use of the unit other than for the purpose intended
 - operation with faulty safety equipment or improperly installed or non-functioning safety and protective precautions
 - unauthorised structural alterations to the unit.

3.4 General information on the safe use of this machine

- Only use the machine together with an extractor.
- Keep your workplace clean. Untidiness increases the risk of accidents.
- For your own safety, please use the accessories and materials recommended in the operating instructions.
- Before servicing the machine, always remove the power supply.

3.5 Disposal

Note:

This symbol indicates, products marked with it should not be disposed off together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through Dentaureum or the dental trade.

3.6 Packing

The packing provides only limited protection against wet, heat and any mechanical force. The packing should be disposed of in the proper manner.

3.7 Delivery

- 1 Sandblasting Machine Dentablast 230 V – REF 094-530-00; 115 V – REF 094-531-00
- 1 Operating instructions and declaration of conformity
- 1 Metal sandblasting nozzle x 4 mm REF 094-532-00
- 1 Mains cable
- 1 Compressed air tube
- 1 Connection attachment
- 1 Foot control

4. Areas of use and description

4.1 Function

The sandblasting machine Dentablast is a circulatory sandblasting (vacuum blaster) machine for universal application. It is suitable for all conventional abrasive material up to a grain size of 250 µm (with reduced pressure up to 1000 µm). This is suitable for cleaning all types of regular cast objects within the dental laboratory. Other qualities include:

- Easy handling
- Doesn’t take up much space
- High safety due to robust construction
- Friendly service

5. Installation

5.1 Positioning the machine

The machine can be placed onto a normal laboratory workbench.

5.2 The power supply

- The voltage information on the machine type sign must correspond with the mains voltage.
- This machine operates on a 230 V power supply. (the 110 V machine must be connected to a 110...115 V power supply)
 - The socket must be secured with a fuse.

Caution: All electrical servicing either on the socket or the supply line must be carried out by an electrician.

5.3 The compressed air supply

This machine can be connected to a compressor or a compressed air unit. The amount of pressure must not exceed max. 10 bar. The consumption is 150 – 270 l/min.

5.4 Connection to an extraction unit

The machine is supplied with an extension socket for a 230 V extraction unit, which runs automatically when the machine is in use. The extraction is essential for work involving a large amount of dust accumulation, eg. Devesting and cleaning. The machine’s max. performance of 1000 W must not be exceeded. An extractor is not suitable as the filter capacity is too low. The connection outlet on the sandblaster has a diameter of x 40 mm. From here the sandblasting machine can be connected to an external extraction unit.

Note If an extraction unit is used, the dust filter must not be inside the sandblasting machine.

6. Operating the sandblasting machine Dentablast (Fig. 1)

6.1 General information

Caution: The machine must only be used when the sandblaster itself and the sandblasting material have reached room temperature. Otherwise there is a danger of condensation and subsequently lumpy sand.

6.2 Setting the blasting pressure

The Sandblasting machine is equipped with a pressure adjustment dial (black dial). The pressure can be set by pulling and turning the dial. The set pressure is shown on the pressure display.

6.3 Re-filling the abrasive material

The sandblasting machine is filled through the frontal flap. The fill level is between 3 and 4.5 litres.

Note Fill only with the additional use of a sieve, in order to prevent possible impurities from entering the abrasive material and subsequently causing a blockage.


6.4 Sandblasting

Hold the object to be sandblasted in the chamber under the sandblasting head and use the foot control to begin sandblasting. By letting the foot control go, the compressed air is halted and the sandblasting stops.

6.5 Emptying the abrasive material

The abrasive material can be emptied by removing the stopper under the machine.

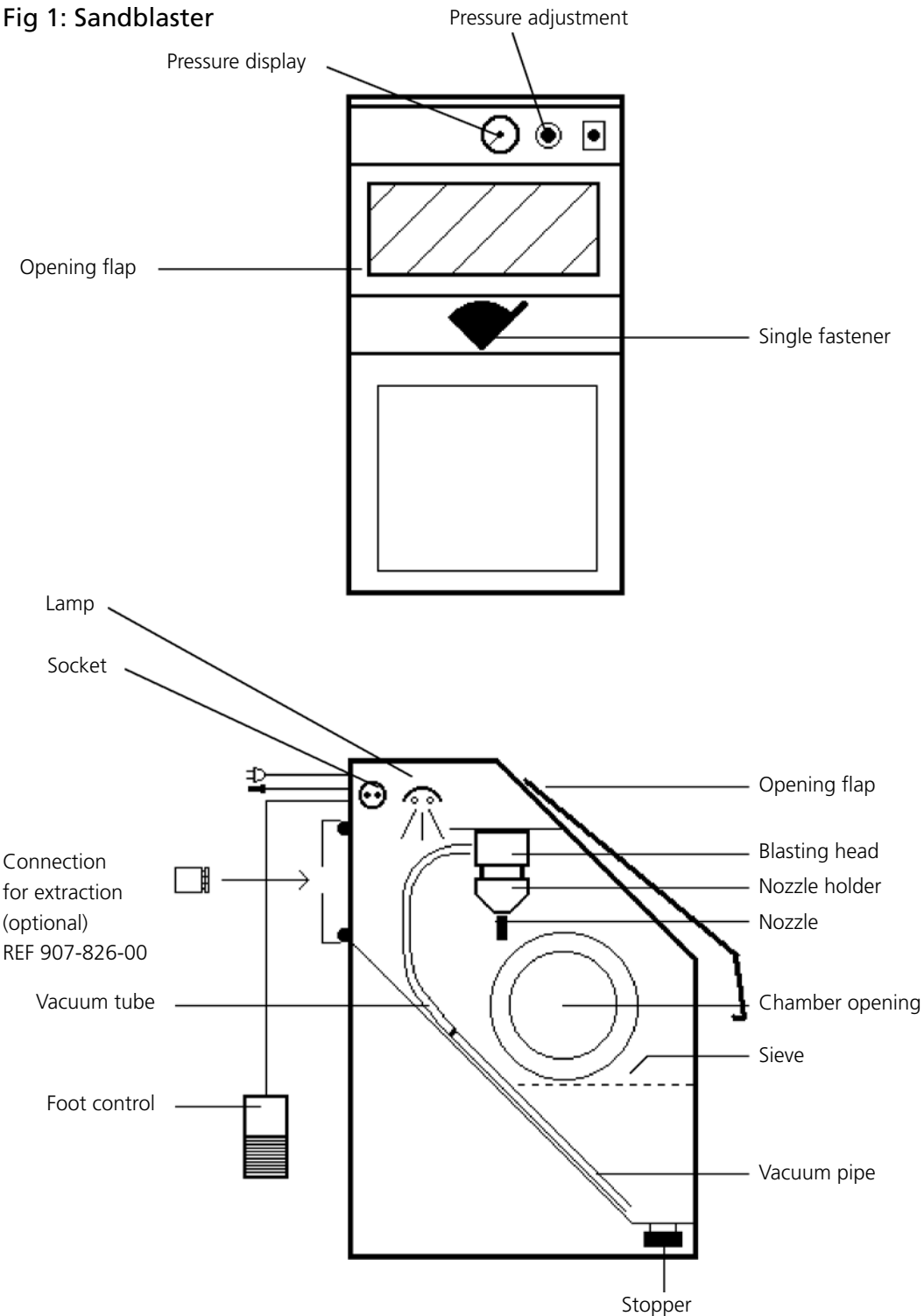Fig 1: Sandblaster

7. Maintenance

⚠ Warning: Always switch off the machine and remove the power supply prior to all types of servicing and maintenance. If you have any technical questions regarding the machine, please contact our customer support team directly under +49 72 31 / 803 - 211.

7.1 Replacing the gloves

Remove the screws from the insertion ring and pull the glove off from the metal ring. Pull the new glove from the inside towards the outside and over the metal ring. Place the glove with ring back into the chamber opening and attach the insertion ring with the screws lightly tightened. Adjust the glove and then carefully tighten the screws alternately.

7.2 Replacing the sandblasting nozzle

Remove the nozzle holder on the sandblasting head (if necessary block the sandblasting nozzle with an object, not with bare hands, and whilst the sandblasting chamber is closed gently tap the foot control. The nozzle holder should jump off.) Remove the sandblasting nozzle and replace with a new one. Press the nozzle holder back onto the sandblasting head.

7.3 Replacing the fluorescent lamp

Remove the lid by loosening the screws, unscrew the lamp reflector beneath and pull the fluorescent lamp (G23 11 W) from its socket and replace with a new one. Screw the reflector and the lid back on.

7.4 Replacing the viewing window

Unscrew the support splints from the base of the frontal flap. Remove the glass pane and replace with a new one. Screw the support splints back on.

7.5 Replacing the vacuum tube and vacuum pipe

The vacuum tube is connected to the side of the sandblasting head and can be removed by pulling out. Attach the new vacuum tube to the sandblasting head and press on as far as it will go. Place the brass pipe into the tube and pass through to the sieve.

8. Spare parts

Viewing window.....	REF 094-533-00
Fluorescent lamp (G23 - max. 11 Watt)	REF 094-534-00
Metal sandblasting nozzle x 4 mm	REF 094-532-00
Vacuum tube	REF 094-535-00
Vacuum pipe.....	REF 094-536-00
Glove.....	REF 094-537-00

9. Accessories

Extraction unit.....	REF 094-550-00
Blasting medium, white: aluminium oxide 99,6 %:	
Grain size 50 (44 – 74 µm) – 4.5 kg	REF 128-017-00
Grain size 100 (88 – 125 µm) – 4.5 kg	REF 128-020-00
Grain size 250 (210 – 297 µm) – 10 kg	REF 128-016-00
Blasting medium, brown:	
Grain size 250 (210 – 297 µm) – 10 kg	REF 128-011-00
Polishing beads	
Grain size 75 (50 – 105 µm) – 4.5 kg	REF 128-211-00
Connection for extraction	REF 907-826-00

10. Technical data

Voltage	REF 094-530-00 – 230 V; REF 094-531-00 – 115 V
Frequency	50/60 Hz
Average power consumption	30 W
Compressed air connection	0...10 bar
Air consumption per hour	270 l at 10 bar; 150 l at 5 bar
Measurements [width x height x depth]	330 x 570 x 410 mm
Weight	15 kg

11.

EC-Declaration of Conformity

DENTAURUM GmbH & Co. KG
Turnstr. 31
75228 Ispringen

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

Description of unit:	Dentablast REF 094-530-00 / 094-531-00	
Type of unit:	sandblast unit	
Start with Unit No.:	125-001 126-001	
EC guidelines:	Guideline for machine	89/392/EEC
	Electromagnetic compatibility	89/336/EEC
	Electrical equipment used within certain voltage limits	73/23/EEC
	Modified Guideline	93/68/EEC
Applied unified standards:	IEC 601-1-1-3	

Date and manufacturers signature: 01.01.2010
Signatory:

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