Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo

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Contents

1	Declaration of Conformity	11
2	Safety instructions	12
2.1	Correct use	12
2.2	Hazard warning symbols	12
2.3	Warranty and liability	12
2.4	Owner responsibility	12
2.5	Personnel responsibility	13
2.6	General advice on operating safety	13
2.7	Significant dangers	13
2.8	Disposal	13
2.9	Packing	13
2.10	Delivery	13
3	Installation	14
3.1	Positioning the machine	14
3.2	Connecting the water supply to the high frequency casting machine <i>megapuls compact</i>	14
3.3	Connecting the power supply	15
3.4	Filling of the water tank	15
4	Operation	15
4.1	Initial operation	15
4.2	Day to day operation	15
5	Maintenance	16
5.1	Cleaning the unit	16
5.2	Emptying the water tank	16
6	Correcting faults	17
7	Technical data	17

1 Declaration of Conformity

EEC-Declaration of Conformity

Dentaurum GmbH & Co. KG

Turnstr. 31

D 75228 Ispringen

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EEC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

Description of Machine Table with water circulation unit

REF 090-606-00

From machine No. 112-0001

EC-Guidelines	EC-Machinery directive	98/ 37/EEC
	EC Low voltage guidelines	73/ 23/EEC
	EC Guideline Electromagnetic compatibility	89/336/EEC

Applied unified standards	EN 292-1/2
	EN 50081-1

Date / Signature of manufacturer 26.01.2006

Personal details to signatory

D
DENTAURUM
GmbH & Co. KG
Turnstraße 31 • 75228 Ispringen • Germany
Telefon +49 7231 / 803-0 • Fax +49 7231 / 803-295

.....
- Dipl.-Ing (FH) K.Merkle (Engineer)-
Productionmanager

2 Safety instructions

Caution Read through the instructions for installing and operating the unit carefully before switching it on.

2.1 Correct use

The table with water circulation unit enables the high frequency casting machine **megapuls compact** to be operated without connecting it directly to the water supply. It also serves as a support table for the casting machine, positioning it at an optimum working height.

The unit is intended solely for use with the high frequency casting machine **megapuls compact** in a dental laboratory. The unit should only be used for the purpose for which it is intended. Dentaureum GmbH & Co. KG, is not liable for damages resulting from any other use of the machine. Approved use also includes adhering to the instructions for use as well as servicing and carrying out maintenance work at regular intervals.

When a Dentaureum product is finally no longer in use the disposal regulations of that particular country apply. Dentaureum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

2.2 Hazard warning symbols

In the instructions for use, the following warning symbols were used:

Warning Suggests possible danger to life or health of persons.

Ignoring this warning may result in serious harm to health or even fatal injury.

Caution Suggests a possibly dangerous situation.

Ignoring this warning may result in slight personal injury or damage to equipment .

Note General information regarding the machine.

2.3 Warranty and liability

Our general terms for sale and delivery apply. Warranty and liability to persons or machinery is not applicable when attributable to one or more of the following points:

- incorrect operation, use, assembly and maintenance of the machine
- misuse of the machine
- operation of the machine despite defect safety fixtures or incorrectly fitted or faulty safety and protective precautionary measures
- failure to follow the advice in the instructions for use regarding transport, storage, assembly, operation and maintenance of the machine
- failure to monitor working parts
- personally conducted alterations to the build of the machine
- unauthorized repairs

2.4 Owner responsibility

The owner may only permit those persons to use the machine who

- have been trained to use the machine and are familiar with the safety precautions
- have read and understood the safety advice and instructions for use
- have been instructed in the relevant accident prevention precautions

2.5 Personnel responsibility

Before use, all persons wishing to operate the machine are obliged to

- abide by the standard instructions regarding working safety
- read and understand the safety precautions and instructions for use and sign in confirmation

2.6 General advice on operating safety

- do not make any alterations to the machine
- only operate the unit when it is filled with water
- only use the machine if it is in perfect working order
- keep your workplace clean. Chaos can increase the possibility of accidents
- for your own safety use only materials and accessories which are recommended in the instructions for use. The use of other materials can increase the possibility of injury for the operator and is regarded as misuse
- always make sure the machine is switched off and the mains plug has been pulled out before maintenance

2.7 High risks

- check the water level once a week. Operating the unit when the water level is too low can damage the pump
- switch off the unit when the casting machine is not in use.

2.8 Disposal

Note

This symbol indicates, products marked with it should not be disposed off together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through Dentaurum or the dental trade.

2.9 Packing

The packing protects the unit during transit, but provides only limited protection against wet, heat and any mechanical force. The packing should be disposed of in the proper manner.

2.10 Package contents

- | | | |
|-----|--|----------------|
| 1 | Table with water circulation unit for high frequency casting machine megapuls compact | REF 090-620-00 |
| 1 | Instructions for use with EC Declaration of Conformity | |
| 1 m | Water pressure hose for connecting to the high frequency casting machine megapuls compact | |
| 1 m | Water outlet hose for connecting to the high frequency casting machine megapuls compact | |
| 1 | Hose clamp | |

3 Installation

3.1 Positioning the machine

The unit should be placed on a solid base and set level. It should be installed in a place which is clean and as free from dust as possible.

⚠ Caution A minimum gap of 20 cm should be left between the table with water circulation unit connected to the casting machine and any units on either side. This ensures that the casting machine is well ventilated.

3.2 Connecting the water supply to the high frequency casting machine *megapuls compact*

Connect the water pressure hose with the $\frac{3}{4}$ " angled connector (Fig. 1, Pos. 2), which is attached to the back of the water circulation unit (Fig. 1), to the "WATER IN" connection on the high frequency casting machine *megapuls compact* (Fig. 2).

Connect the water outlet hose without the angled connector (Fig. 1, Pos. 3) to the "WATER OUT" connection on the high frequency casting machine *megapuls compact* (Fig. 2) and secure the hose with the hose clamp supplied.

Ensure that the screw on the hose clamp is properly tightened.

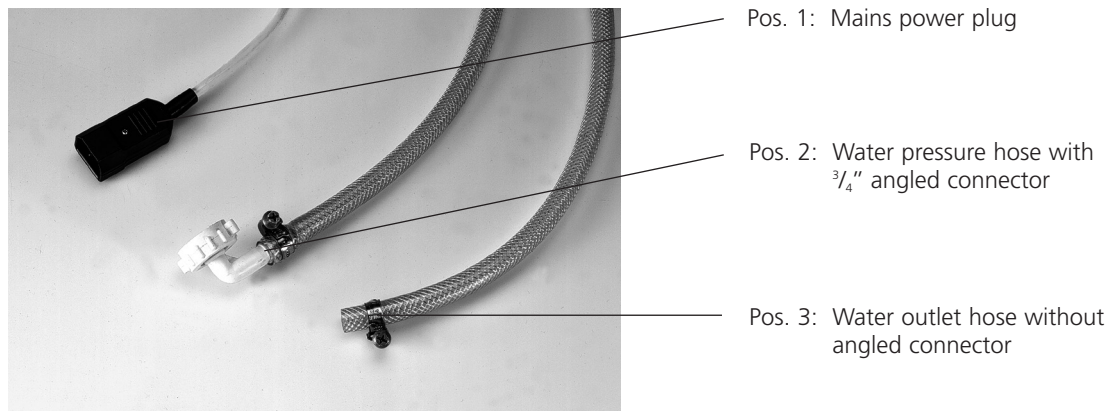
Fig. 1: Hose connectors

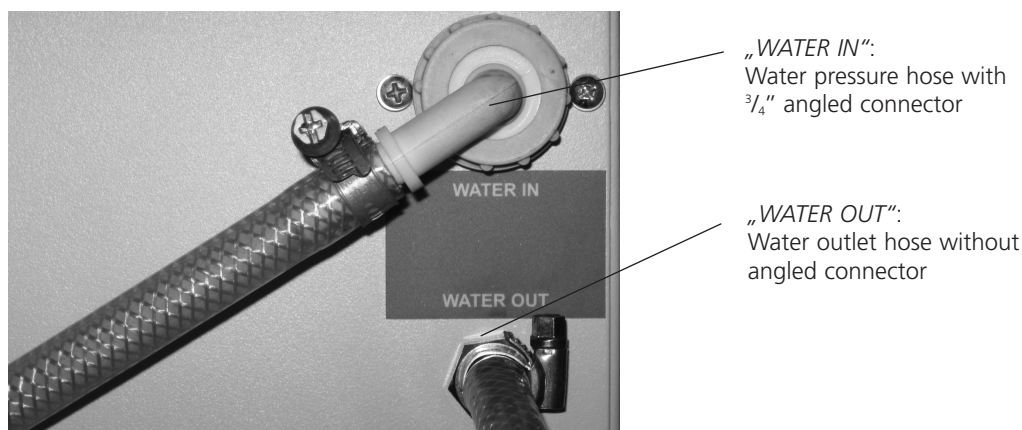
Fig. 2: Connection to the high frequency casting machine *megapuls compact*

3.3 Connecting the power supply

The unit should be connected to the 230 V socket on the back of the high frequency casting machine **megapuls compact**. This ensures that the water circulation unit is always switched on or off at the same time as the casting machine.

- ⚠ Caution** Any maintenance work at the plug socket or mains supply may only be carried out by an electrician.
- ⚠ Caution** Always connect the electricity cable of the water circulation unit to the socket of the casting machine.

3.4 Filling the water circulation unit with water

Recommendation Use distilled water to avoid lime deposits.

- ⚠ Caution** Connect the water circulation unit to the high frequency casting machine **megapuls compact** before filling it with water. Take out the plug of the water circulation unit.

Pull the plug from the high frequency casting machine **megapuls compact**.

Switch off the water pump and take out the plug. Slowly pull out the drawer with the water tank, which is at the front of the unit, until the water inlet stopper is accessible (Fig. 3). Remove the stopper and, using a funnel, pour the water into the inlet to approx. 1 cm from the top of the tank (approx. 20 litres). Replace the stopper. Plug in the unit.

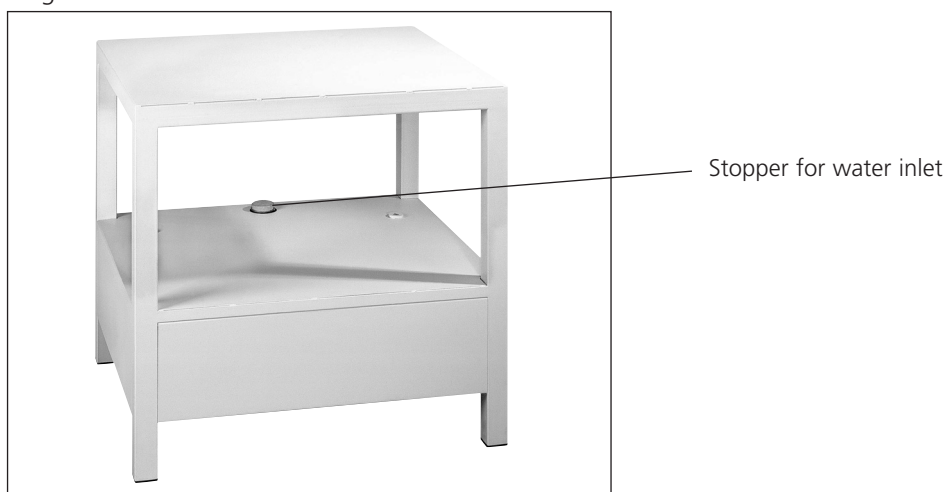
Fig 3: Drawer with water inlet

4 Operation

4.1 Initial operation

After the unit has been filled and switched on for the first time, check that water can be seen flowing through the transparent water hoses. If larger bubbles of trapped air can be seen, refer to section 6, Correcting faults.

4.2 Normal operation

The pump of the water rotating unit is switched on by the switch on the high frequency casting machine **megapuls compact**.

When the casting machine is switched off after casting, the electricity supply is automatically cut off to the water circulation unit. The water circulation unit should, however, still be switched off at the ON/OFF switch.

- i Note** To avoid overheating of the device a break of 10 minutes must be maintained after having made a series of 7 castings.

5 Maintenance

 **Warning** Always disconnect the unit from the mains during any servicing, cleaning and maintenance work.
There should always be two people with electrical experience present when servicing and repairs are being carried out.
All work on the electrical components of the unit should only be carried out by a qualified electrician or by Dentaurem customer services.

 **Caution** Check the water level regularly. If the water level is too low, the casting machine is no longer cooled properly. If the tank is empty, the pump can be damaged.

5.1 Cleaning the unit

Wipe the unit occasionally with a damp cloth. Do not use abrasive cleaning agents or tools.

5.2 Emptying the water tank

Switch the casting machine off.

Unscrew the water pressure hose with angled connector („Water IN“) and hold the hose in a container into which the water can be drained.

 **Caution** Tank holds 20 liters.

Switch the casting machine on and let it run until the pump of the water circulation unit becomes louder.

Completely drain the rest of the water without using the pump.

 **Warning** Do not use compressed air to remove the remaining water.
This may damage the tank.

6 Correcting faults

Error	Reason	Correction
Red "ERROR" indicator on the casting machine megapuls compact lights up, audible signal sounds.	Water flow interrupted. No electricity. The water tank has not been mit filled with enough water.	Check hose connection between megapuls compact and water rotating unit. Connect water rotating unit to the socket on the reverse side of megapuls compact . Check fuse (1,6At) at the reverse side of the water rotating unit. Check the water level and add water if necessary.
Pump becomes noisy, air bubbles can be seen in the transparent hoses.	Air in the water circulation.	Switch off the water circulation unit and casting machine. Unscrew the pressure hose with angled connector ("WATER IN") from the casting machine and hold it in a container. Turn on the casting machine until water starts running out of the hose. Switch off casting machine and reconnect hose. Switch on the casting machine and check hose connections for leakage. Add water if necessary.

If you are not able to correct the fault using this table, please call your relevant service technician or call Dentaureum Technical Service – Machines directly.

Tel. +49 72 31 / 803 - 211 | info@dentaureum.com

7 Technical data

Mains voltage	260 V
Mains frequency	50/60 HZ
Output	260 W
Current	1.1 A
Electric fuse	1.6 At
Capacity	20 l
Pump output	9 l/min
Max. water pressure	3.3 bar
Ambient air temperature for operation	15...40 °C / 59...104 °F
Storage temperature	5...40 °C / 41...104 °F
measurements [width x height x depth]	680 x 650 x 600 mm
Net weight approx.	41 kg

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[illegible]

This image shows a full page of white paper with horizontal dotted lines. The lines are evenly spaced and run across the width of the page, providing a guide for handwriting practice. There are no margins, text, or other markings on the page.

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.

DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

- ➔ Informationen zu Produkten und Serviceleistungen finden Sie unter www.dentaurum.com
- ➔ For more information on our products and services, please visit www.dentaurum.com
- ➔ Vous trouverez toutes les informations sur nos produits et services sur www.dentaurum.com
- ➔ Descubra nuestros productos y servicios en www.dentaurum.com

Stand der Information | Date of information | Mise à jour | Fecha de la información: 2022-11

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications
Reservado el derecho de modificación

D
DENTAURUM